


## INSTRUCTIONS FOR USE



For allergic emergencies (anaphylaxis)

Read these Instructions for Use carefully before you use EpiPen<sup>®</sup> or EpiPen Jr<sup>®</sup>. Before you need to use your EpiPen<sup>®</sup> or EpiPen Jr<sup>®</sup>, make sure your health care professional shows you the right way to use it. Parents, caregivers, and others who may be in a position to administer EpiPen<sup>®</sup> or EpiPen Jr<sup>®</sup> Auto-Injector should also understand how to use it as well. If you have any questions, ask your health care professional.

### Your EpiPen<sup>®</sup> and EpiPen Jr<sup>®</sup> Auto-Injector

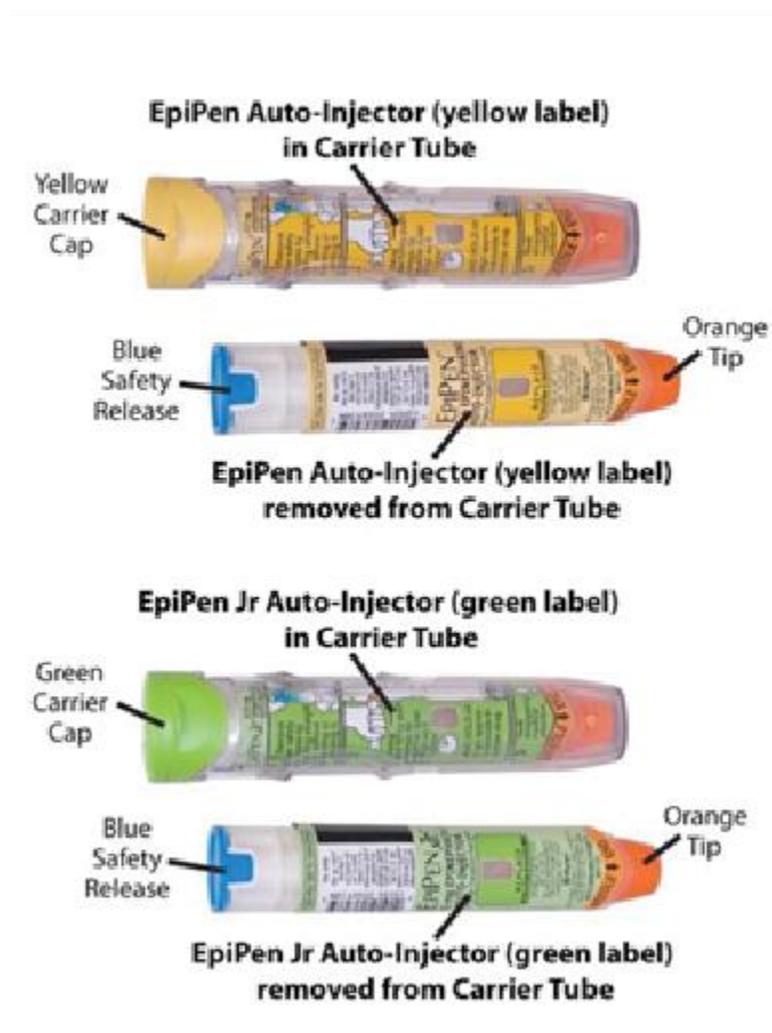

A dose of EpiPen<sup>®</sup> or EpiPen Jr<sup>®</sup> requires 3 simple steps:  
Prepare, Administer and Get emergency medical help

### Step 1. Prepare EpiPen® or EpiPen Jr® for injection



#### Remove the EpiPen® or EpiPen Jr® from the clear carrier tube.

Flip open the yellow cap of your EpiPen® or the green cap of your EpiPen Jr® carrier tube.



Tip and slide the auto-injector out of the carrier tube.



**Grasp the auto-injector in your fist with the orange tip pointing downward.** With your other hand, **remove the blue safety release by pulling straight up** without bending or twisting it.

#### Note:

- The needle comes out of the orange tip.
- To avoid an accidental injection, never put your thumb, fingers or hand over the orange tip. If an accidental injection happens, get medical help right away

### Step 2. Administer EpiPen® or EpiPen Jr®



Hold the auto-injector with orange tip near the middle of the outer thigh (upper leg).

**Swing and firmly push the orange tip against the middle of the outer thigh** until it 'clicks'. Keep the auto-injector firmly pushed against the thigh at a 90° angle (perpendicular) to the thigh.



**Hold firmly against the thigh for approximately 10 seconds** to deliver the medicine. The injection is now complete.



**Remove the auto-injector from the thigh.** The orange tip will extend to cover the needle.



Massage the injection area for 10 seconds.

**Step 3. Get emergency medical help now.** You may need further medical attention. You may need to use a second EpiPen® or EpiPen Jr® Auto-Injector if symptoms continue or recur.

- Take your used auto-injector with you when you go to see a health care professional.
- Tell the health care professional that you have received an injection of epinephrine. Show the health care professional where you received the injection.
- Give your used EpiPen® or EpiPen Jr® Auto-Injector to the health care professional for inspection and proper disposal.
- Ask for a refill, if needed.

Note:

- The used auto-injector with extended needle cover will not fit in the carrier tube.
- EpiPen® or EpiPen Jr® are single-use injectable devices that deliver a fixed dose of epinephrine. The auto-injector cannot be reused. It is normal for most of the medicine to remain in the auto-injector after the dose is injected. The correct dose has been administered if the orange needle tip is extended and the window is blocked.
- Your EpiPen® or EpiPen Jr® Auto-Injector may come packaged with an EpiPen® Trainer and separate Trainer Instructions for Use. The EpiPen® Trainer has a grey color. The grey EpiPen® Trainer contains no medicine and no needle. Practice with your EpiPen® Trainer, but always carry your real EpiPen® or EpiPen Jr® Auto-Injector in case of an allergic emergency.
- Do not try to take the EpiPen® or EpiPen Jr® Auto-Injector apart.

This Patient Information and Instructions for Use has been approved by the U.S. Food and Drug Administration.

Manufactured for:

Mylan Specialty L.P., Morgantown, WV 26505, U.S.A. by Meridian Medical Technologies, Inc., Columbia, MD 21046, U.S.A., a Pfizer company

EpiPen® and EpiPen Jr® are registered trademarks of Mylan Inc. licensed exclusively to its wholly-owned affiliate, Mylan Specialty L.P. of, Morgantown, WV 26505, U.S.A.

© 2014 Mylan Specialty L.P. All rights reserved.

Revised: April 2014

EPIP-N:R00

3/15 EPI-2015-0151



Register your EpiPen® or EpiPen Jr® Auto-Injector at [MyEpiPen.com](http://MyEpiPen.com) and find out more about:

- Free EpiPen® Auto-Injector Refill Reminder Program. It is important to keep your auto-injector up-to-date.

Register up to 6 EpiPen® or EpiPen Jr® Auto-Injectors and receive automatic Refill Reminder Alerts.

- Receive periodic information related to allergies and allergens.

For more information about EpiPen® or EpiPen Jr® Auto-Injectors and proper use of the product, call Mylan at 1-877-446-3679 or visit [www.epipen.com](http://www.epipen.com).